

Hematologies, Inc.

Shipping Instructions

1. Complete Requisition, Attach Billing & Diagnostic Information.



2. Seal Specimen in Biohazard Bag. Place Specimen in Shipping Container and Close.



3. Place Completed Requisition with Billing Information in Second Biohazard Bag & Seal.



4. Place Iceberg Cool Pack on Top of Sealed Biohazard Bag and Close Shipping Container.



5. Place Shipping Container in FedEx Diagnostic Pack & Seal. Complete and Attach Airbill.
**If Shipped on Friday
Please Mark for Saturday
Delivery and place Saturday
Delivery Sticker on Envelope.**



Call FedEx for Pick-Up (800-463-3339)
Call **Hematologies** Client Services Dept. with Airbill Number
(800-860-0934)